



BRAVO[™] vacuum autoclave





Your Infection Control Specialist™



The BRAVO vacuum chamber autoclave

The BRAVO fractionated vacuum chamber autoclave continues the SciCan tradition of being at the forefront of innovation and technology. BRAVO incorporates both pre-vac (to remove air) and post-vac (to help drying) cycles to ensure perfect steam penetration for any load. The BRAVO is true to the STATIM[®] reputation for speed, providing rapid cycles for all types of instrument loads.

FEATURES

Speed & Capacity

The BRAVO line of chamber autoclaves is one of the fastest on the market today. With the BRAVO's speed and drying technology, your daily output of reprocessed instruments cannot be rivaled.

The BRAVO's large chamber capacity,

with a rotating rack, can accommodate five DIN trays that can hold up to 25 pouches**, three full size wrapped cassettes, or a mixture of both.



Versatility

The BRAVO offers numerous sterilization cycles with a wide array of parameters. This allows the user to choose the appropriate cycle for the load, be it wrapped or unwrapped.

Filling & draining

The BRAVO chamber autoclave has a number of filling and draining options.

The BRAVO includes a top filling port, providing users with a third water filling option. This is in addition to the conveniently located front filling port and the rear automatic fill port.

Direct to drain option: Lastly, a rear mounted, direct-to-drain port facilitates automatic drainage eliminating the need to manually drain the reservoir.

> Rear mounted automatic fill port

New top filling port

Rear mounted, direct-to-drain port

SciCan

Clean water exhaust port Direct-Water-Pull Kit, L (01-111774S) e

r t New water Used water exhaust port



Ease of use

Similar to all other SciCan products, the BRAVO is very easy to use. Simply load the unit, choose your cycle and press the start button.



Cycle Recording

For any future guidelines, the BRAVO saves all the data from each cycle. The integrated front USB port allows cycle data to be stored digitally making record keeping simple and efficient. BRAVO units can also be ordered with an integrated printer.

Dryness

The Dri-Tec^v drying system, with closed door drying, gives the BRAVO lightning fast & consistent dry loads every time.





Cycles (134°C @ 4 mins.)	Cycle times (include drying)			Max. Load Mass		
	17	17V	21V	17/17V	21V	
Porous / Wrapped	40 mins.	35 mins.	40 mins.	*	*	
Hollow / Unwrapped	35 mins.	27 mins.	32 mins.	6kg / 13.6lbs	7.5kg / 16.5lbs	
Solid / Wrapped	30 mins.	24 mins.	28 mins.	3kg / 6.6lbs	4.0kg / 8.8lbs	
Solid / Unwrapped	22 mins.	19 mins.	23 mins.	6kg / 13.3lbs	7.5kg / 16.5lbs	

* 17(V)/ 21V: Unpackaged porous materials:1kg / 1.25kg, Porous material in single package: 0.75kg / 1.0kg, Solid material/hollow instruments in single pack: 3kg / 4kg, Solid material/hollow instruments in double package: 1.5kg / 2.0kg.

Model	Dimensions (LxWxH) [†]	Chamber Volume	Chamber Dimension	Model Weight	Voltage
BRAVO 17/17V	22.0 x 19.0 x 16.5" 560 x 480 x 420mm	4.5 gal / 17L	D10 x L13.5" D254 x L343mm	128lbs / 55kg	220 / 230V, 60Hz, 10A, 2300W or
BRAVO 21V	25.0 x 19.0 x 16.5" 635 x 480 x 420mm	5.5 gal / 21L	D10 x L17.5" D254 x L445mm	139lbs / 63kg	220 / 240V, 50Hz, 10A, 2300W

⁺When installed into cabinets, a minimum 4" / 2" (102mm / 51mm) free cool air space is required on the back/sides, and top of the unit.

BRAVO 50Hz		1	BRAVO 60Hz		
M7A300015	BRAVO 17 No printer		M7A300013	BRAVO 17 No printer	
M7A300016	BRAVO 17 With printer		M7A300014	BRAVO 17 With printer	
M7A310015	BRAVO 17V No printer		M7A310013	BRAVO 17V No printer	
M7A310016	BRAVO 17V With printer		M7A310014	BRAVO 17V With printer	
M7A320015	BRAVO 21V No printer		M7A320013	BRAVO 21V No printer	
M7A320016	BRAVO 21V With printer		M7A320014	BRAVO 21V With printer	

 ** Load capacity can vary based on pouches and / or cassettes used.

STATIM[®] G4 Cassette Autoclaves



STATIM 2000 and 5000 G4

Manufactured by: SciCan Ltd. 1440 Don Mills Rd. Toronto, Ontario M3B 3P9 Tel: 416-445-1600 Toll free: 1-800-667-7733 Fax: 416-445-2727



STATIM[®] "Classic" Cassette Autoclaves

STATIM 2000 and 5000



Your Infection Control Specialist™

EU Representative: SciCan GmbH, Wangener Strasse 78, 88299 Leutkirch Germany Tel.: +49 (0)7561 98343 - 0 Fax: +49 (0)7561 98343 - 699 (Office for Swiss customers) SciCan Medtech AG Alpenstrasse 16 6300 Zug SWITZERLAND Tel.: +41 (0) 41-727 7027 Fax.: +41 (0) 41-727 7029

HYDR/M[®] M2 Washer-disinfector



FOLLOW US ON:



