Implante Escleral "Esnoper"

Scleral Implant "Esnoper"

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Trapezoidal design which makes it stable and easy to implant

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Prolonges the aqueous humour drainage in non perforating filtration surgery in open angle glaucoma

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2 holes, one for fixation to the sclera and the other to improve the aqueous humour filtration

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Longitudinal irrigation channels to maximize the effect of the aqueous humour fluid

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Snoper device is suitable for non perforating surgery in open angle glaucoma. It is implanted under scleral flap and it intends to create a permanent intraescleral lake to improve the aqueous humour filtration in patients suffering from open angle glaucoma

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Biocompatibility and comfort for the patient

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Creates a permanent intra-scleral lake

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Prevents adhesions between the scleral flap and the trabecular meshwork

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Special design which allows its insertion into the suprachoroid space to improve the filtration of the aqueous humour

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A very thin superficial scleral flap of 4,5 x 4,5 mm should be shaped at 40% depth.

A deeper flap is made under the first one, 4 x 4 mm limbus based, leaving just a thin underlying scleral flap, forming the scleral lake.

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In the anterior limit of the scleral lake, we find the peripheral cornea and the roof of the Schlemm's Canal which must be resected, as well as the underlying juxtacanalicular trabecular meshwork taking caution not to perforate it. A proper humour percoration is achieved without the need of penetrating into the anterior chamber.

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The V2000 implant is placed into the scleral lake and it is fixed with a 10/0 nylon suture.

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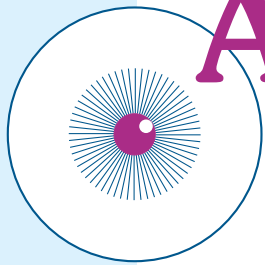
(*). As optional step, in order to maximize the filtration of the aqueous humour and prior to the fixation of the implant to the sclera, the surgeon may perform an incision on the scleral lake in order to insert the posterior side of the V-2000 into the suprachoroid space.

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The conjunctiva is finally closed with the same type of Nylon.

Parque Tecnológico de Alava
01510 MIÑANO - ALAVA (Spain)
Telfs. 945 29 82 56 - 945 29 82 89
Fax 945 29 82 09
E-mail: ajlsa@ajlsa.com
www.ajlsa.com

AJL, S.A.