

MENICARE PURE

the natural solution for cleaner, healthier lenses.



Part of the Menicon RGP Care Solution Family.
Crafted protection for more comfortable,
more hygienic, and cleaner lenses.



**For healthier eye care,
you need the Menicon RGP care solution family**

MENICARE PURE - pure and natural eye comfort

Use daily for cleaning, disinfecting, and rinsing your lenses. It's the natural solution for cleaner, healthier lenses.

For healthier eye care, you need the Menicon RGP Care Family.

Basic combination for all users, MENICARE PURE multi-purpose solution, pure and natural eye comfort.

- Daily-use lens care solution
- Use for cleaning, disinfecting, rinsing, and storage



MENI CARE PURE

Additional comfort for RGP contact lens wearers

Natural Formulation

- Polylysine - natural homopolymer with antimicrobial properties
- Vitamin C glucoside - natural antioxidant which is believed to have anti-inflammatory and collagen production properties
- Hyaluronic acid - natural moisturizing ingredient
- Macrogolglycerol (HCO60-) - has a good balance between less cytotoxicity (less irritation) and higher cleaning efficacy for lipids. Also macrogolglycerol has the ability to prevent lysozyme from denaturing
- HPMC - a high molecular cellulose derivative made from naturally existing cellulose fiber, enhancing the solution viscosity whilst providing a comfortable cushioning effect on insertion.

Proven efficiency

- Proven to disinfect against bacteria, yeast and moulds
- Effectively cleans away protein and lipid deposits

Pure

- Extremely low toxicity - safe to use
- Dual filtration eliminates microorganisms and insoluble particles



MENI CARE PURE

Product profile

Indications

- Cleaning
- Rinsing after cleaning
- Disinfecting - effective within 5 minutes
- Soaking
- Rinsing before insertion and lubricating

Composition

- ϵ - Polylysine: 0.01%
- Edetate disodium: 0.05%
- HCO-60: 0.1%


Directions




1. Wash hands thoroughly before handling contact lenses.




2. Rub the lens carefully with several drops of MENICARE PURE solution for 20 seconds.

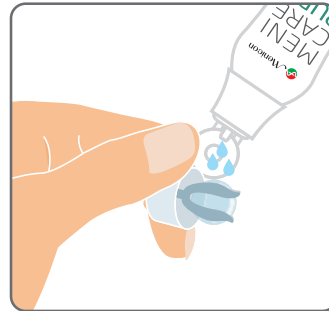

3. Place the lenses in the Right and Left lens holder and rinse them with MENICARE PURE.

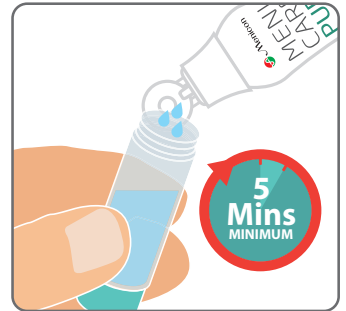

4. Fill the lens case with MENICARE PURE and soak the lenses for a minimum of 5 minutes or overnight.

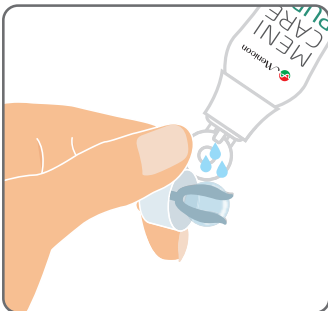

5. Before insertion, rinse the lenses with MENICARE PURE.